Gebrauchsinformation

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Instructions

Application

DuraClean² is specially formulated to keep your CLEARsplint[®] flexible nightguard fresh and clean. Its advanced, alcohol free formula is alkaline based and free of harmful chemicals that could damage your nightguard.

Directions

Using the enclosed nightguard box, dissolve one scoop (3/4 tsp) of DuraClean² in ½ cup of warm tap water.

Mix solution.

Soak nightguard in solution for 15-20 minutes.

Brush nightguard thoroughly before rinsing. Rinse nightguard under tap water.

Also rinse nightguard box, sink & counter from any excess solution.

Store nightguard in a dry place or in dry nightguard box. Keep jar tightly closed when not in use.

CAUTION! Please note the following:

- Box with child-proof lock
- For opening press cover outbox and turn clockwise
- Do not use in mouth
- Never swallow or gargle solution
- Keep away from children
- If powder/solution comes in contact with eyes-rinse thoroughly
- In case of spill on skin, counter, sink or clothes- rinse immediately with water.

Packaging:

REF 261705 Set cleaner (170 g) with box and brush

REF 26171 cleaner 170 g

REF 26172 cleaning brush

REF 26173 cleaning box

The product was developed for use in the dental field and must be used in accordance with instructions.

For any damage arising from misuse or improper application, the manufacturer accept no liability.

The user agrees to check the product before use on suitability.

Particularly if these purposes are not in the leaflet.

Date of Information: Februar 2013

Kentzler-Kaschner Dental GmbH, Mühlgraben 36, 73479 Ellwangen/Jagst,
Telefon +49 (7961) 9073-0, Fax +49 (7961) 52031 – www.kkd-topdent.de