MUPPY

Zungengitter starr

D
GB
F
I
E

Patienteninformation

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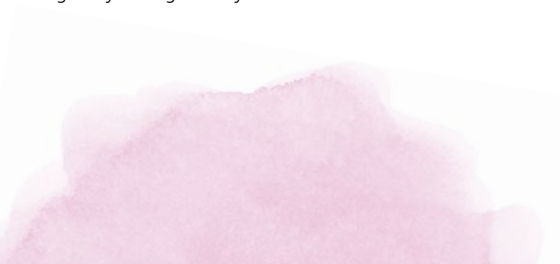
D
GB
F
I
E

Patient information

MUPPY for treating a frontal open bite

In principle, once the cause is weaned off, the frontal open bite is the only anomaly that through self-healing, finds its way back to the normal overbite by keeping the tongue away. This also applies to the mixed teeth when the open bite is accompanied by a missing tooth position. The treatment is named "interceptive therapy" because its success is based on the weaning of the habit. In such cases, **MUPPY** must also be worn regularly during the day – often over several months.


MUPPY-rules

Studies have shown that more than 90 % of children give up damaging sucking habits with the help of **MUPPY**. Your support and a few rules help your child:

- 1 **MUPPY** is a substitute for thumb and dummy. The substitution must be voluntary.
- 2 **MUPPY** must always be worn at night.
- 3 To get used to it easier children should also wear **MUPPY** a few hours during the day (when playing or watching television).
- 4 As a rule, weaning is successful within a week. However, **MUPPY** should also be worn for 6–8 weeks after successful weaning of the dummy ("recidivism").
- 5 **MUPPY** can be stored in the hygiene box.
- 6 Do not forget to regularly brush the teeth and **MUPPY**!

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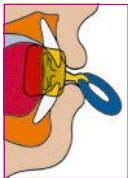MUPPY is used similarly to a common weaning dummy. The shield is placed between the lips and teeth. The tongue crib lies behind the teeth and prevents the tongue from positioning itself between the rows of teeth.

If your child is having speech therapy treatment, you should take note of the following:

- Follow your speech therapist's individual wearing and exercise instructions exactly.
- Support your child during this stage, as he/she has to feel the importance of mouth closure in order to take the exercises seriously.
- Make your child aware of the "rules of play" and supervise the necessary exercises.

Instructions for use

Before using **MUPPY** for the first time, clean it thoroughly under warm running water. Check **MUPPY** regularly for places that have been bitten through! Replace **MUPPY** when necessary.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the OROFAN® dental cleaning tablets. **MUPPY** is neither dishwasher-safe nor microwave-safe. Only clean the hygienic **MUPPY**-Box with a damp cloth.

MUPPY designs

MUPPY wire guard is available in two different sizes:

- with violet ring for deciduous dentition (size I)
- with blue ring for mixed dentition (size II)

MUPPY wire guard rigid
(for milk teeth)

MUPPY wire guard rigid
(for mixed teeth)

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